ARTISAN®/ARTIFLEX® Enclavation Needles

EN

>> INSTRUCTIONS FOR USE

STERILE EO

CE

0344

Description

The ARTISAN®/ARTIFLEX® Enclavation Needle is a disposable medical device, consisting of a holder and 2 needles. The needles have preformed curves and a nail like tip. The holder carries a left and a right curved needle. The Enclavation Needles are used for fixation of the ARTISAN®/ARTIFLEX® Intraocular lenses for the correction of aphakia or for the refractive correction in a phakic eye. The Enclavation Needle is used to form an 'iris bridge' facilitating the enclavation of iris tissue in the 'claws' of the ARTISAN®/ARTIFLEX® lenses.

Warnings & precautions

- Do not resterilize or reuse;
- Sterility and product functionality cannot be guaranteed after cleaning/resterilization;
- Do not use after the expiration date;
- Do not use if the integrity of the product or the packaging has been compromised;
- Store in a dry place at room temperature;
- A high level of skill and training specific to iris-fixated IOLs is required for ARTISAN®/ARTIFLEX® lens implantation.

Operating instructions

- Make a fold of iris tissue under the claws;
- Slightly lift the fold of iris and place it through the claw by lifting one claw arm;
- The resilience of the claws will automatically grasp the iris, when the fold has passed through the claws;
- The hooklike cannula should be retracted carefully, avoiding any hooking onto the iris surface.

The manufacturer shall not be liable for any injury or damage suffered by a patient due to use of the product.

ARTISAN®/ARTIFLEX® Enclavation Needle

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OPHTEC USA, Inc. 6421 Congress Ave. Suite 112 • Boca Raton FL 33487 • USA